



Connecting a Lab to your HeronCloud Account

1 Go to HeronCloud.com and create an account

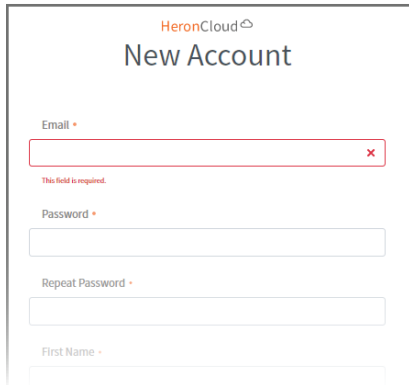


After you submit the form you will receive an email with a link to activate your account


2 After activating your account, please login and click on the connections link

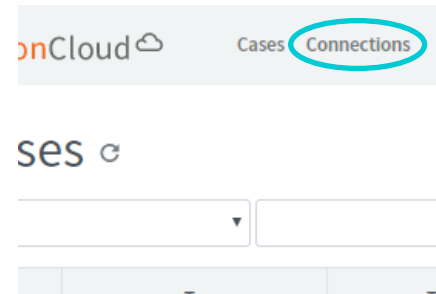

3 Click on Add Connection

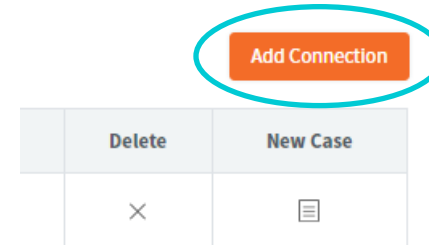


4 Complete the form for the lab that you would like to connect to, and click the send button once completed

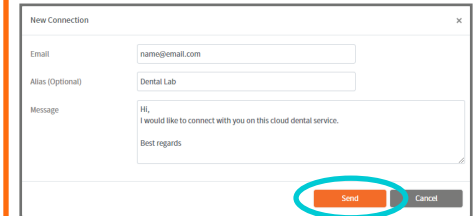


To complete the connection, the New Connection must create their own HeronCloud account

5 Once a Connection is made you will have to login to the HeronCloud through QuantorClinic in order to select a lab for an order, this is done through the settings


To get to the settings, click the settings button located at the top right of QuantorClinic

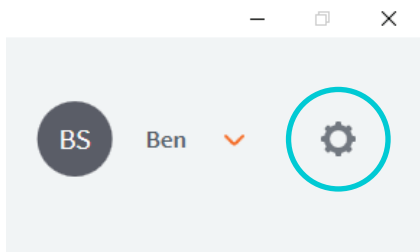


6 Once in settings, navigate to the Users section and select the icon to the right of the user that you would like to connect to the HeronCloud

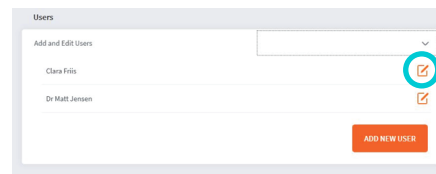

7 On the users information select the HeronCloud button on the bottom left

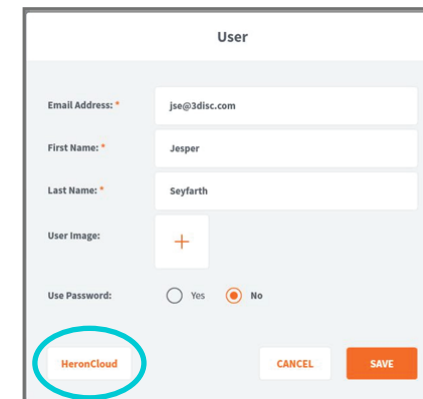

8 With the "Use the HeronCloud.com account" selected click the OK button and login to the HeronCloud with the information you setup in step 1

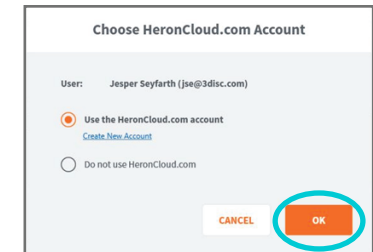

Once logged in you will be able to send files to your connections